



#FeelsParenting.

Safety and cleaning instructions for the LIINI®breast pump

Dear Customer,

Thank you for choosing our breast pump. We place great importance on your safety and the well-being of your family.

To ensure that you can use your device safely, it is important to carefully follow the safety and cleaning instructions below. Improper use can pose serious risks!

Manual / Operating instructions

We deliberately do not provide printed operating instructions, thereby saving several tonnes of paper every year. At the same time, we can update our electronic operating instructions at any time so that you always have the latest version at your fingertips.

Even though many functions can be operated intuitively, please take **five minutes** to **read through the manual**. It is important to use the product in accordance with the operating instructions so that you can enjoy it for a long time, everything works safely and unnecessary annoyance is avoided.

You can find the **operating instructions** via the following **QR code** (scan with your smartphone):



Available online: DE / EN / FR / IT

Bankverbindung CHF:
IBAN: CH06 0021 0210 1087 0101 V
BIC / SWIFT: UBSWCHZH80A

Bankverbindung EUR:
IBAN: CH23 0021 0210 1087 0160 X
BIC / SWIFT: UBSWCHZH80A

Kontakt
LIINI GmbH
Bahnhofstrasse 21
4571 Lüterkofen
Web: www.LIINI.ch
E-Mail: info@LIINI.ch
Tel: +41 79 559 06 02

Rechtliches
UID: CHE-131.522.878 MWST
Steuernr. DE: 09417/18455
Umsatzsteuer-ID: DE352084049
EORI-Nr: DE905090565529397
Zollkonto ZAZ: 19867-7



#FeelsParenting.

General information

Manufacturer	LIINI GmbH Bahnhofstrasse 21 4571 Lüterkofen Switzerland
Contact	compliance@liini.ch
Country of origin	China (CN)
Certifications	CE, RoHS
EU authorised representative	ProductIP AR Services Galvanistraat 1 6716 AE Ede The Netherlands (authorities only)
WEEE registration number (DE)	74241330 WEEE Register Nordostpark 72 90411 Nuremberg
WEEE registration number (AT)	9008392342860 Federal Ministry for Climate Protection, Environment, Energy, Mobility, Innovation and Technology (BMK) Radetzkystraße 2 1030 Vienna
WEEE registration number (LU)	MA1403 ecotrel asbl 11, Boulevard du Jazz 4370 Belvaux,
WEEE registration number (NL)	42125 Stitching OPEN Baron de Coubertinlaan 7 2719 EN Zoetermeer

Bankverbindung CHF:
IBAN: CH06 0021 0210 1087 0101 V
BIC / SWIFT: UBSWCHZH80A

Bankverbindung EUR:
IBAN: CH23 0021 0210 1087 0160 X
BIC / SWIFT: UBSWCHZH80A

Kontakt
LIINI GmbH
Bahnhofstrasse 21
4571 Lüterkofen
Web: www.LIINI.ch
E-Mail: info@LIINI.ch
Tel: +41 79 559 06 02

Rechtliches
UID: CHE-131.522.878 MWST
Steuernr. DE: 09417/18455
Umsatzsteuer-ID: DE352084049
EORI-Nr: DE905090565529397
Zollkonto ZAZ: 19867-7



#FeelsParenting.

General safety instructions (GPSR)

Carrying your baby incorrectly can lead to serious accidents and injuries and poses **serious safety risks**.

Please observe the following instructions carefully:

- Only use the LIINI® breast pump in accordance with these instructions.
- The product is intended for its intended purpose only – expressing breast milk.
- The breast pump is not a toy and must not be used or cleaned by children.
- Check all components for damage before each use. Do not use the device if any parts are damaged, deformed or incorrectly assembled.
- Only use the components and attachments supplied or intended by LIINI®.
- Stop using the device immediately if you experience pain, discomfort or unusual reactions.

Instructions for use

- Ensure that the breast pump is correctly assembled before use.
- Always select an intensity level that is comfortable for you. Excessive suction can be uncomfortable.
- The breast pump must not be used while sleeping.
- Do not use the device while driving or performing activities that require your full attention.
- The breast pump is not waterproof. Keep the electrical unit away from water and moisture.

Battery and charging instructions

- Only charge the breast pump via the designated **USB-C port**.
- Only use suitable power adapters with the appropriate output power.
- Do not charge the device unattended or near heat sources.
- Do not use the breast pump while it is charging.
- Do not expose the battery to extreme temperatures, direct sunlight or open flames.

Bankverbindung CHF:
IBAN: CH06 0021 0210 1087 0101 V
BIC / SWIFT: UBSWCHZH80A

Bankverbindung EUR:
IBAN: CH23 0021 0210 1087 0160 X
BIC / SWIFT: UBSWCHZH80A

Kontakt
LIINI GmbH
Bahnhofstrasse 21
4571 Lüterkofen
Web: www.LIINI.ch
E-Mail: info@LIINI.ch
Tel: +41 79 559 06 02

Rechtliches
UID: CHE-131.522.878 MWST
Steuernr. DE: 09417/18455
Umsatzsteuer-ID: DE352084049
EORI-Nr: DE905090565529397
Zollkonto ZAZ: 19867-7



#FeelsParenting.

Cleaning instructions

- Clean all parts that come into contact with breast milk before first use and after each use.
- Completely disassemble the breast pump before cleaning.
- Parts that come into contact with milk (e.g. silicone cushions, valves, breast milk containers) can be cleaned with warm water and mild detergent, boiled or sterilised with other baby utensils (according to standard recommendations).
- The electrical unit must not be immersed in water or cleaned under running water.
- Clean it only with a dry or slightly damp cloth.
- Allow all components to dry completely after cleaning before reassembling or storing them.

Storage

- Store the breast pump in a clean, dry place.
- Store the device out of the reach of children.
- Do not expose the product to prolonged moisture or direct sunlight.

Bankverbindung CHF:
IBAN: CH06 0021 0210 1087 0101 V
BIC / SWIFT: UBSWCHZH80A

Bankverbindung EUR:
IBAN: CH23 0021 0210 1087 0160 X
BIC / SWIFT: UBSWCHZH80A

Kontakt
LIINI GmbH
Bahnhofstrasse 21
4571 Lüterkofen
Web: www.LIINI.ch
E-Mail: info@LIINI.ch
Tel: +41 79 559 06 02

Rechtliches
UID: CHE-131.522.878 MWST
Steuernr. DE: 09417/18455
Umsatzsteuer-ID: DE352084049
EORI-Nr: DE905090565529397
Zollkonto ZAZ: 19867-7



#FeelsParenting.

Information on the WEEE Directive 2012/19/EU

Meaning of the crossed-out wheelie bin symbol



All our electronic devices (bottle warmers, baby food warmers, sleep aids) are marked with the crossed-out wheelie bin symbol.

The crossed-out wheelie bin symbol means that **you are legally obliged** to dispose of these devices separately from unsorted municipal waste, i.e. old devices must be disposed of separately from household waste. Non-permanently installed batteries or rechargeable batteries must be removed beforehand and disposed of separately.

All rechargeable batteries in our products are permanently installed, so you can **dispose of** the products **as a whole**.

A bar under the wheelie bin means that the product was placed on the market after 13 August 2005.

Options for returning old appliances (WEEE Directive 2012/19/EU)

Owners of old appliances can return them to the return or collection points set up and made available by public waste disposal authorities to ensure proper disposal.

Personal data (WEEE Directive 2012/19/EU)

All end users of waste electrical and electronic equipment are advised that they are responsible for deleting personal data on the waste equipment to be disposed of.

Bankverbindung CHF:
IBAN: CH06 0021 0210 1087 0101 V
BIC / SWIFT: UBSWCHZH80A

Bankverbindung EUR:
IBAN: CH23 0021 0210 1087 0160 X
BIC / SWIFT: UBSWCHZH80A

Kontakt
LIINI GmbH
Bahnhofstrasse 21
4571 Lüterkofen
Web: www.LIINI.ch
E-Mail: info@LIINI.ch
Tel: +41 79 559 06 02

Rechtliches
UID: CHE-131.522.878 MWST
Steuernr. DE: 09417/18455
Umsatzsteuer-ID: DE352084049
EORI-Nr: DE905090565529397
Zollkonto ZAZ: 19867-7



#FeelsParenting.

Information on the Battery Directive 2006/66/EC

Whenever reference is made to batteries or accumulators below, this refers to waste batteries or accumulators.

Meaning of the crossed-out wheelie bin symbol (Battery Directive 2006/66/EC)

The crossed-out wheelie bin symbol means that you are legally obliged to dispose of batteries or accumulators separately from unsorted municipal waste, i.e. they must be disposed of separately from household waste. If batteries or accumulators contain more than 0.0005% mercury by weight, more than 0.002% cadmium by weight or more than 0.004% lead by weight, they are marked with the chemical symbols of the metals (Hg, Cd, Pb) for which the limit value is exceeded.

Options for returning batteries or accumulators (Battery Directive 2006/66/EC)

Owners of batteries or accumulators can return them free of charge after use to the return or collection points set up and made available by public waste disposal authorities or directly to retailers to ensure proper disposal.

Waste prevention measures and measures to prevent littering (Battery Directive 2006/66/EC)

Reduce battery waste as much as possible by using batteries with a longer service life or rechargeable batteries. Do not simply leave batteries or battery-powered electrical and electronic equipment lying around and thus avoid littering the environment - for the protection of our children.

Options for preparing used batteries for reuse (Battery Directive 2006/66/EC)

Check the possibility of recycling used batteries.

Potential effects of the substances contained in batteries on the environment and human health (Battery Directive 2006/66/EC)

Batteries and accumulators may contain substances that have a potentially negative impact on human health and the environment. Lithium-containing batteries in particular must be handled with care due to the specific risks they pose, such as fire hazard or risk of swallowing in the case of button cells.

Importance of separate collection and recycling (Battery Directive 2006/66/EC)

The separate collection and recycling of batteries and accumulators make an important contribution to reducing the burden on the environment and preventing health hazards.